


## **2X-CEL Chamber (80 micron) Instructions for use**

1. Clean the 2X-CEL chamber and coverslip with kimwipe. (Do not use alcohol.)
2. Place the Magnetic Slide Holder on a slide warmer.
3. Place the 2X-CEL chamber in the center of the Magnetic Slide Holder.
4. Position the coverslip so that both sides of the 2X-CEL chamber are covered (see reverse side for illustration).
5. Carefully close the magnetic clasp of the holder.
6. Load sample by placing a small drop of specimen (12-13  $\mu$ l) at the opening of the chamber.
7. Place magnetic holder on the IVOS or microscope stage and analyze as usual.

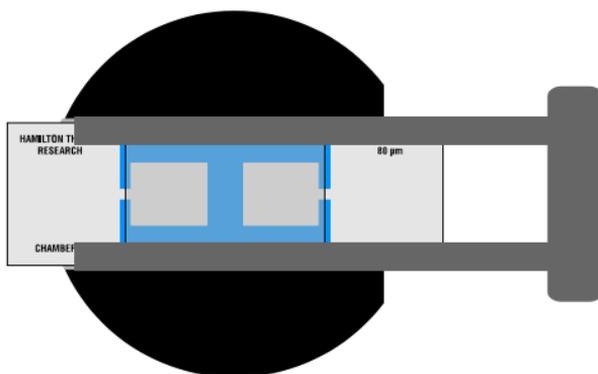

2X-CEL Chamber (80 micron) properly positioned in Magnetic Slide Holder.